
Fluor Protector Gel

To strengthen the teeth. With calcium, fluoride and phosphate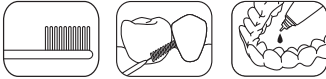

English

To strengthen the teeth
With calcium, fluoride and phosphate

Description: Fluor Protector Gel is a gel for intensive and at the same time gentle dental care. Calcium, fluoride and phosphate protect the teeth and repair and harden stressed dental enamel. The provitamin D-Panthenol benefits the gingiva. Xylitol inhibits the growth of bacteria causing caries in the oral cavity.

Recommended application: For the daily special care to protect teeth from caries, erosions and periodontal diseases. Application is particularly suitable with sensitive teeth, exposed cervicals, acidic diet, dry mouth, during orthodontic treatment, after professional tooth cleaning and in the course of a tooth whitening procedure.

Depending on your needs, you may apply Fluor Protector Gel:

– instead of tooth paste 2-3 times daily for 3 minutes using a tooth brush

– in the evening after tooth brushing with the usual tooth paste. Apply Fluor Protector Gel with a tooth brush.

– to clean the space between teeth or fixed restorations – for example bridges or implants. Use Fluor Protector in conjunction with an interdental brush for cleaning.

– by filling the gel in your tray and let it react for 10 minutes each 1-2 times daily.

Tip: Do not rinse after use! In this way, the effect of the Fluor Protector Gel is enhanced.

Note:

- Contains potassium fluoride.
- For children younger than 6 years of age: Use only pea-sized amounts of gel. In order to prevent excessive swallowing, tooth brushing should be supervised. If fluoride is additionally administered, ask your physician or dentist.

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Ivoclar Vivadent AG, Bendererstrasse 2, FL-9494 Schaan, Liechtenstein